



BIO MARKING DYES

Markers for surgical margins



CODE	DESCRIPTION	PACKAGING	UDI-DI
05-014-030-1	30 ml bottle with dropper, blue	1 x 30 ml	08034120276245
05-015-030-1	30 ml bottle with dropper, black	1 x 30 ml	08034120276276
05-016-030-1	30 ml bottle with dropper, green	1 x 30 ml	08034120276306
05-017-030-1	30 ml bottle with dropper, yellow	1 x 30 ml	08034120276337
05-018-030-1	30 ml bottle with dropper, orange	1 x 30 ml	08034120276368
05-019-030-1	30 ml bottle with dropper, red	1 x 30 ml	08034120276399
05-020-030-1	30 ml bottle with dropper, purple	1 x 30 ml	08034120276429



In vitro Diagnostic – Medical Device
 IVD Class A, Reg. UE 2017/746
 Basic UDI: 080341202W01030799BM



Manufacturer: Bio-Optica Milano S.p.A.

Date of issue: 03/11/2022
 Rev. 002



GENERAL INFORMATION

Dyes for identification and orientation of excised surgical specimens formulated to define margins without bleeding, changing colour or fading and without staining fixatives and solvents used for tissue processing. Bio Marking Dyes are a-toxic, made of polymers of natural origin.



INSTRUCTIONS

1. Shake the bottle before use;
2. For fresh specimens (not fixed yet), apply the dye on the tissue, directly from the bottle or with the help of a brush. **LET DRY FOR 2-3 MINUTES AND DIX IN FORMALIN;** For fixed specimens, remove the excess of fixative from tissue surface and apply the dye like on fresh specimen;
3. **The dyes bind to the tissue surface at room temperature (15-30°C) in 2-3 minutes;**
4. At the end of the use, close the bottle to avoid evaporation.

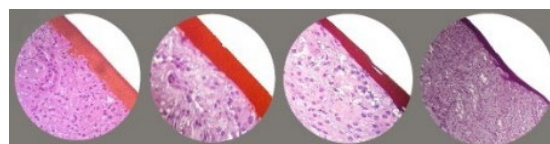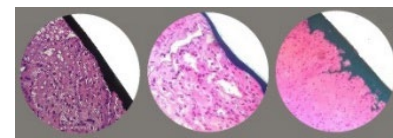

Technical details

Specifications	Expected aim	Product for the preparation of cyto-histological samples for optical microscopy. Marking of surgical margins.	
		Drying time at room temperature:	2-3 minutes
	Packaging	Primary container: container in white PP. Secondary container: carton box. Lable: Wear, water, alcohol and solvent resistant PVC label. Scratchproof ink resistant to water and alcohol.	
Storage	Storage	Store the preparation at 15-25°C. Keep the containers tightly closed.	
	Storage temperature	15-25°C	
	Stability	After opening, it is reusable until the expiry date, if correctly stored.	
	Validity	1 year	
Warning	Product classification	The product is intended for professional laboratory use for healthcare professionals. Carefully read the information on the label (danger symbols, risk and safety phrases) and always consult the safety data sheet. Do not use if the primary container is damaged. In the event of a serious accident, we recommended that you immediately inform Bio-Optica Milano S.p.A and the competent authorities.	
	Disposal	Waste according to MSDS, observe all state and local environmental regulations regarding waste disposal.	

REVISION n°	REASON	REVISION DATE
001	Regulation adjustment UE 2017/746 - IVDR	16/05/2022
002	Code review	03/11/2022

Date of issue: 03/11/2022
Rev. 002