

HISTOCOLD - Dispenser

*Storage system for a safe transport of histological specimens
From the surgery room to the PA lab
(In conformity with the European Regulation 605/2014)*



CODE	DESCRIPTION	PACKAGING
05-100200	HistoCold dispenser	1 pz



In Vitro Diagnostic – medical device



Manufacturer: Bio-Optica Milano S.p.A.

Date of issue: 23/10/2018
Rev. 01

Dispenser

Refrigerating dispenser of small dimensions for the dispensing of HistoCold solution at 4°C.
The instrument is provided with a pedal for an easy use.

Instructions for use

- 1- Open the HistoCold tank by removing the blue lid;
- 2- Screw the aspirating tube of the dispenser on the HistoCold tank;
- 3- Push the pedal to spill the refrigerated solution.

Technical specifications

Dimensions	l	325 mm
	d	340 mm
	h	1230 mm
	Weight	26 kg
Characteristics	Power supply	220 – 240 V
	Electrical power	180 W

Date of issue: 23/10/2018

Rev. 01